




QUIDEL®

Virena®

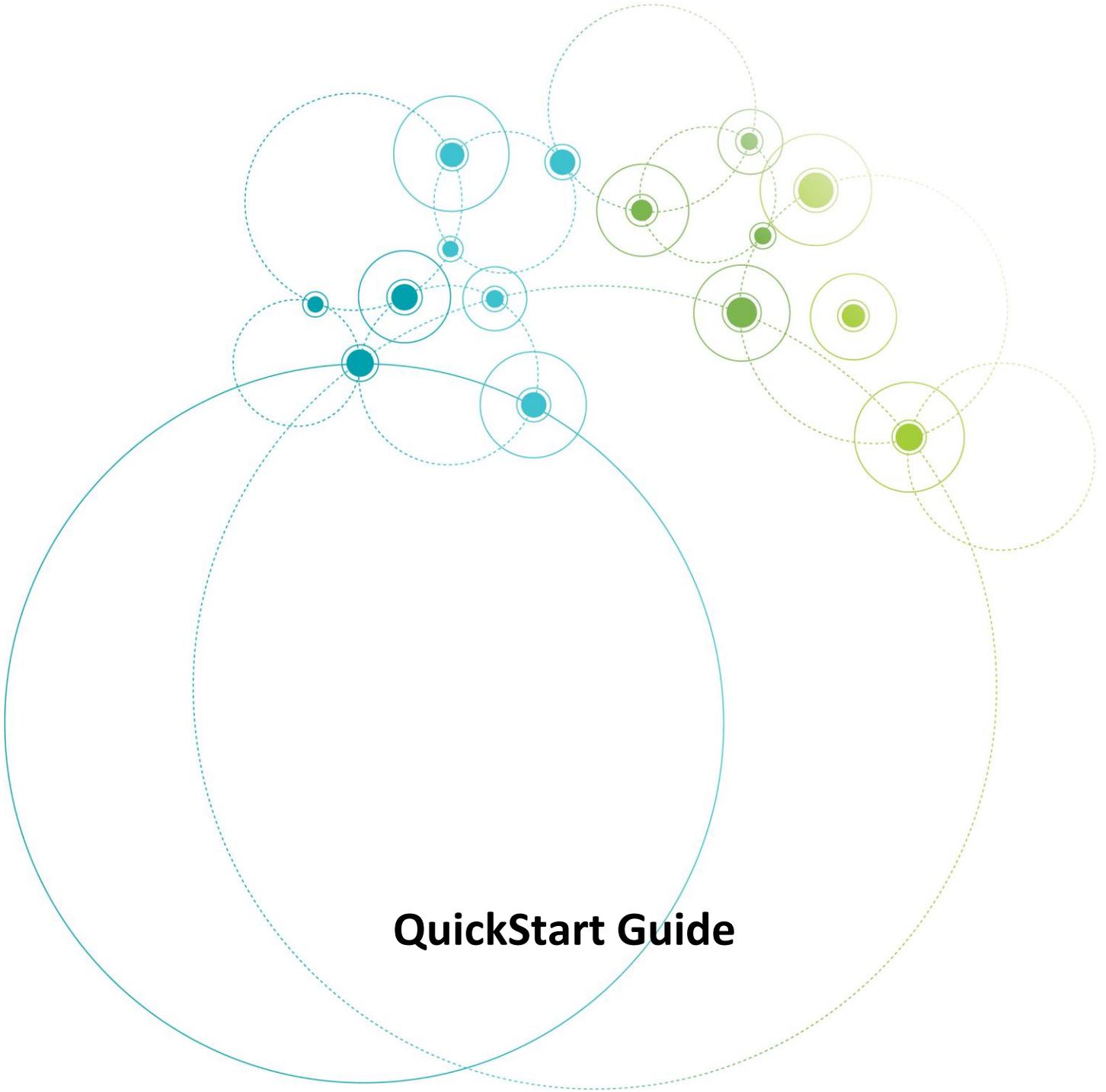

**QuickStart Guide**


# Unpack the Router Box




x1



x2

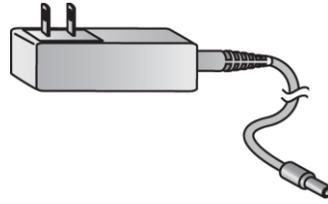

x1

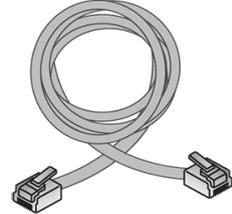

x1

## 1. Install Router and Power On

Connect antennas to router



Connect power supply by rotating the barrel plug clockwise




Verify Connection

- POWER: solid GREEN light
- SIGNAL: solid GREEN light indicates good connection. Amber color indicates weaker connection
- SERVICE: light should flash in groups of 1, 2, or 3



## 2. Connect Router to Sofia

Use Ethernet cable to connect from the LAN port of the router to the LAN port on the back of Sofia

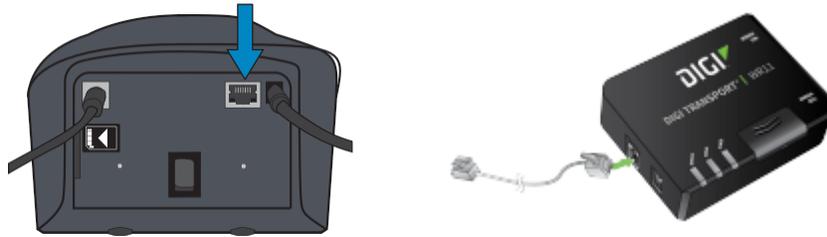


## 3. Check Network Configuration on Sofia

Power ON Sofia.

On Sofia go to Main Menu ► Supervisor Menu (log in) ► Settings ► Network Settings ► Set Network Configuration

Ensure that the screen matches the screen below.

When complete, press “OK” ► Back to return to Network Settings

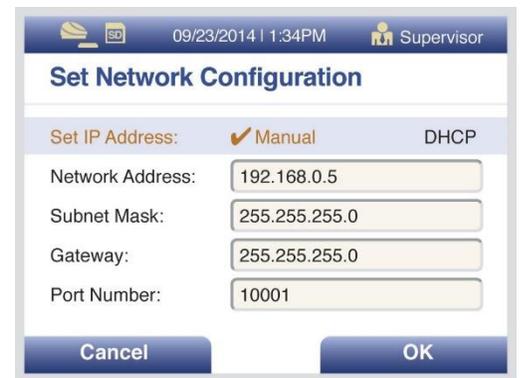
The screenshot shows a mobile application interface for 'Set Network Configuration'. At the top, there is a status bar with a signal strength icon, 'SD', the date and time '09/23/2014 | 1:34PM', and a user icon labeled 'Supervisor'. The title 'Set Network Configuration' is centered. Below the title, there are two radio buttons: 'Manual' (selected with a checkmark) and 'DHCP'. The configuration fields are: 'Network Address:' with the value '192.168.0.5', 'Subnet Mask:' with '255.255.255.0', 'Gateway:' with '255.255.255.0', and 'Port Number:' with '10001'. At the bottom, there are two buttons: 'Cancel' on the left and 'OK' on the right.

## 4. Check Surveillance Mode Set Up on Sofia

On Sofia go to Main Menu ► Supervisor Menu (log in) ► Settings ► Network Settings ► Surveillance Settings

Ensure the screen matches the screen below.

Return to the Main Menu  
(Press “OK” ► Back ► Back)

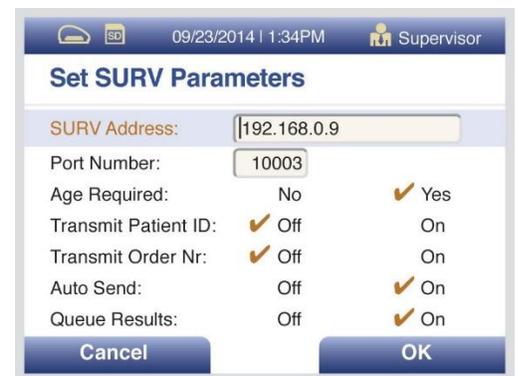The screenshot shows a mobile application interface for 'Set SURV Parameters'. At the top, there is a status bar with a signal strength icon, 'SD', the date and time '09/23/2014 | 1:34PM', and a user icon labeled 'Supervisor'. The title 'Set SURV Parameters' is centered. Below the title, there are two input fields: 'SURV Address:' with the value '192.168.0.9' and 'Port Number:' with '10003'. Below these are four rows of settings, each with a checked radio button and two options: 'Age Required:' with 'No' selected and 'Yes' available; 'Transmit Patient ID:' with 'Off' selected and 'On' available; 'Transmit Order Nr:' with 'Off' selected and 'On' available; and 'Auto Send:' with 'Off' selected and 'On' available. The last row, 'Queue Results:', has 'Off' selected and 'On' available. At the bottom, there are two buttons: 'Cancel' on the left and 'OK' on the right.

## 5. Verify Router Connection

On Sofia go to Main Menu ► Review Data  
Send Results ► Send Results to SURV Receiver ►  
Test SURV Connection

Return to the Main Menu  
(Press "OK" ► Back ► Back)

Next, run a Calibration Cassette to begin sending  
Data to the cloud. (For instructions on running a  
calibration, refer to the Sofia User Manual.)

Return to the Main Menu  
(Press "OK" ► Back ► Back)

If connection is established, screen displays:  
"The Analyzer can be connected to SURV."



Return to Main Menu; Sofia is now ready to  
be run in Surveillance Mode

If connection is NOT established, screen displays:  
"The Analyzer cannot be connected to SURV."



Refer to the detailed Sofia Surveillance  
Installation Instructions for more details

## 6. Operator Number Set Up (Optional)

On Sofia go to Main Menu ► Supervisor Menu ►  
Manage Users

1. Set Operator # to Required and press "OK."
2. Select Options key to Add or Edit User.
3. Enter User ID (barcode scanner can be used).
4. Arrow down to User Type and select Operator or Supervisor.
5. Enter the Operator #\* (different than User ID) and press "Save & Next."
6. Enter User Name and press "OK."

\*Operators may use their USER ID with an extra  
Number at the end for OPERator #.

The screenshot shows the 'Add New User' screen. At the top, it displays the date and time '09/23/2014 | 1:34PM' and the user role 'Supervisor'. The main title is 'Add New User'. Below the title, there are three input fields: 'User ID' with an asterisk, 'User Type' with a dropdown menu currently set to 'Operator', and 'Operator #' with an asterisk. At the bottom, there are two buttons: 'Cancel' and 'Save & Next'.

## 7. Performing Tests in Surveillance Mode

On Sofia go to Main Menu ► Run Test

### Start Test Screen

- Enter (User ID, Patient ID, and Order #) if required, then press START TEST.

### Patient Demographics

- Enter the numeric value of the Patient's age.
- Use the arrow keys to select Patient status.
- Press START TEST.

The screenshot shows the 'Patient Demographics' screen. At the top, it displays the date and time '09/23/2014 | 1:34PM' and the user role 'Supervisor'. The main title is 'Patient Demographics'. Below the title, there are several input fields: 'Age' with an asterisk, 'Age in:' with a dropdown menu set to 'Years', 'Patient Status:' with a dropdown menu set to 'In-patient', and four 'Study Code' fields labeled A, B, C, and D. At the bottom, there are two buttons: 'Back' and 'Start Test'.